
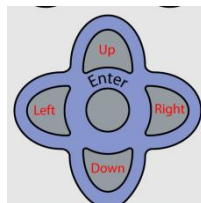



Quick Reference Guide


Clarity Acuity can be used with the supplied remote control or the keyboard. These quick-reference tables show the functions of each of the buttons and keys. The basic functions of the remote are annotated here in red.

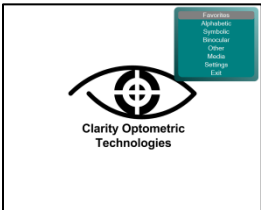

Main Menu	Remote	Keyboard	Function
	Up	Up	Move Selection Up
	Down	Down	Move Selection Down
	Enter	Enter	Choose Selection
	Menu	ESC	Hide Menu (standby)

Charts	Remote	Keyboard	Function
	Up	Up	Increase Acuity/Size
	Down	Down	Decrease Acuity/Size
	Left/Right	Left/Right	Switch Modes *
	Enter	Enter	Randomize Symbols
	Menu	ESC	Return to Menu

* When switching modes, pausing briefly will select to current mode.

Media Viewer	Remote	Keyboard	Function
	Enter	Enter	Play / Pause
	Menu	ESC	Return to Menu

Use Page Up/Down to cycle through your Workflow, F1-F6 to access Hotkeys, and the P key to activate/deactivate Pointer mode.